### Indicazioni:

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### Indications:

**Benexè® Physiological Saline Solution** is recommended for the daily wash or toiletry of the nose and the eyes of the infant, child or adult.

**Nasal use:** **Benexè® Physiological Saline Solution** makes easier the wash of the nose of the babies which is usually obstructed by mucus. It helps getting rid of the secretions. Thanks to the moisturizing action it relives quickly the nasal dryness by the adult.

**Ophthalmic use:** **Benexè® Physiological Saline Solution** can be used for the eye hygiene of the baby through ocular instillation, baths or application of the product on a cotton wool pad. The saline solution can also be used by the infant or adult.

### Forms and Presentations:

Pack of 30 x 5 mL single-dose vials

### Composition:

Sodium chloride: 0.9 g

Purified water qs 100 ml

### Observations:

**Benexè® Physiological Saline Solution** can be used as often as necessary without any hazards over long periods of time. In case of treatment with a nasal or ophthalmic medicine, saline solution must be used before the taking of the medicine and not after.

### Precautions for use:

Single use only.

#### Not for injection.

Keep out of the reach of children.

Do not use after the expiry date.

Do not use the same unit dose for the toiletry of the nose and the eye.

Storage at room temperature.

Opened single-dose vials must not be reused, since the product does not contain preservatives.

As this medical device is not containing any preservative, a re-use would lead to the risk of administration of a contaminated solution.

### Directions for use:

Break off a single-dose vial and open it by twisting the top off.

**Nasal use:** introduce the tip of the container into the nostril, the head slightly inclined, and gently press the unidose vial. Then blow the baby's nose. For baby, instill the product with a minimum of pressure in order to avoid any risk of middle ear contamination.

**Ophthalmic use:** soak a cotton wool then gently wipe the baby eyelids or pour some drops in the eye and wipe using a sterile compress.

For the adult, the product can be used by instillation, washing or eye-baths.

CE mark certification was obtained on 25/08/2000.

Date of revision of the instructions for use: November 2012.

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**Distributor:** MITI SpA - Corso Italia 22 - 20122 Milano - ITALY