Benexè®

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18 SUPPOSTE - 2500 mg

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18 SUPPOSITORIES-2500 mg

Please read this leaflet carefully before using this product: it answers to some frequently asked questions on Benexè® Supposte di Glicerolo. Use the product according to the instructions provided in the leaflet and in case of doubts ask for clarification to your doctor or pharmacist. Do not throw away this leaflet and keep it together with the product: it may be necessary to read it again.

Product presentation

Benexè® Supposte di Glicerolo is a disposable medical device for rectal use, recommended to facilitate the evacuation activity of the bowel.

Composition

One suppository of 2500 mg contains: glycerol 2315 mg, sodium stearate and sodium bicarbonate.

When the use of Benexè® Supposte di Glicerolo is recommended

Benexè® Supposte di Glicerolo is recommended for treatment of short duration of occasional constipation. It is a suppository with a temporary action, which is able to lubricate fecal mass to promote its ejection. Do not use Benexè® Supposte di Glicerolo for any other purpose. Do not ingest. For rectal use only.

When the use of Benexè® Supposte di Glicerolo is not advisable

Do not use Benexè® Supposte di Glicerolo in presence of cramps, severe abdominal pain of unknown origin and if the patient suffers from intestine inflammatory diseases, as ulcerative colitis or Crohn's disease. Avoid to use the product also in presence of nausea, vomit, blood into the faeces, anal rhagades, intestinal obstructions, Ileus (loss of intestinal muscular tone), low levels of potassium into the blood and severe state of dehydration. Do not use Benexè® Supposte di Glicerolo in case of allergy or hypersensitivity to glycerol or one of the other components. Do not use after the expiry date printed on the box and the aluminium or if the package is damaged.

Inform your doctor if:

- you notice the absence of any intestinal movement, even for few days, or a sudden change of the usual bowel habits, for more than two weeks;
- the product has been ingested by accident;
- undesirable effects occur;
- you are taking other medicines or medical devices.

Use in children

Benexè® Supposte di Glicerolo is not indicated for children under 12 years.

Use during pregnancy or breast-feeding

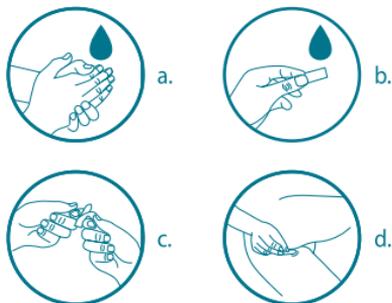
There are not known interactions with other active pharmaceutical ingredients, therefore, in the absence of clinical data and in case of concomitant use with other medicines or medical devices, it is recommended to consult your doctor before using Benexè® Supposte di Glicerolo.

Activity and properties

Glycerol-based preparations intended for rectal use show an evacuant activity without the need of product absorption by the intestinal mucosa. Glycerol, indeed, when inserted into the rectum is capable to promote the evacuation by a physical mechanism which does not involve a systemic absorption. Glycerol softens the fecal mass and lubricate rectum wall, causing intestinal movement and muscular contractions that promote the fecal mass ejection.

Instructions for use:

Accurately wash the hands before and after the use of suppositories (a). If the suppository is too soft put it for some minute in a refrigerator or under cold water (b). Remove the external aluminium (c) and wet the suppository with cold water or aqueous lubricant to facilitate the insertion. Lie down on the side and insert the sharp side of the suppository into the rectum. Gently push the suppository completely with a finger: otherwise, it could be expelled (d). Remain lying down, possibly on your belly, for some minutes before to get up. Benexè® Supposte di Glicerolo should have effect within thirty minutes.

FOR RECTAL USE ONLY.

When and for how long use

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It is advisable to use Benexè® Supposte di Glicerolo once a day. Consult your doctor in case of lack of results or if constipation persists. A prolonged use of laxatives (more than 7 consecutive days) is not recommended and could lead to addiction. Excessive doses of the product can cause abdominal pain and diarrhea. Loss of liquids and electrolytes should be reintegrated.

Side effects

A mild rectal burning sensation could be perceived.

Recommendation for the maintenance of a correct intestinal functionality

Drink much water, increase the fibres amount into the diet and regularly perform physical activity contribute to the functionality of the intestine.

Storage and disposal instructions

Keep the product in a cool and dry place, away from direct sunlight and source of heat.

Do not store over 30°C.

Do not disperse into the environment after use.

Keep the product out of reach of children.

Shelf-life

The product has a 5 years shelf-life if correctly stored in its original container.

Product description

Benexè® Supposte di Glicerolo is available in the following packaging and dosage:

18 suppositories of 2500 ml - ADULTS

Date of issues or latest revision of this leaflet:

March 2021.

 Lab. Chim. Farm. A. Sella s.r.l.
Via Vicenza, 67 - Schio (VI) - Italy

MEDICAL
DEVICE

KEY TO SYMBOLS

Important:
read the
instructions
for use

Do not use
more than once

Storage
temperature
below 30°C

Manufacturer

Read the
leaflet carefully
before use

Lot number

Expiry date

Do not
release into
the environment

Catalogue
number

Keep away
from heat