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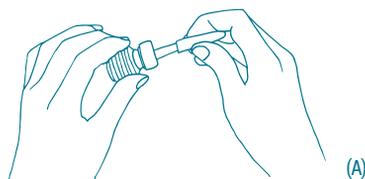
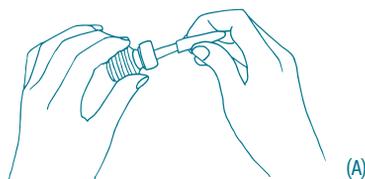
Benexè® Micro-enemas Natural Adults is a CE Medical Device based on Altelax, a polysaccharide complex of Altea and Mallow with evacuating, emollient action and protective of the rectal mucosa. Softens faeces, facilitates evacuation and counteracts discomfort, irritation and inflammation present in case of constipation. Also contains Honey and extracts vegetables such as Lemon and Lavender. Also useful in cases where cleaning of the part is required terminal of the rectum and anus.

Composition: glycerol, demineralized water, Honey, Altelax (polysaccharide complex of Altea e Mallow), Hydroglyceric extract of Lemon, Hydroglyceric extract of Lavender, preservatives: potassium sorbate, sodium benzoate.

How to use: from 12 years old use 1 micro-enema of 9 g as needed. In case of obstinate constipation use 2 doses at the same time. Each single-dose container is individually sealed until the moment of use to ensure hygiene. Open the microenema by removing the cannula cover (A).

Lubricate the cannula with a few drops of product, introduce the cannula into the rectum and press the bellows fully (B).

At the end of the application, keep the bellows pressed until the cannula is completely extracted to prevent the product from being sucked back in. After using the product, avoid contact with the eyes and wash your hands thoroughly.

Warnings: for rectal use only. Do not swallow. In case of persistent stypsis, it is important to consult your doctor to exclude other diseases. Micro enemas should not be used in cases of acute abdominal pain or pain of unknown origin. Keep out of children's sight and reach. Do not use the product after the expiry date reported on the box. Do not use in case of damaged pack. In case of pregnancy or lactation or if a pregnancy is going to be planned, consult a doctor or pharmacist before use. Do not use the product under 12 years old.

Contraindications and side effects: do not use the product in case of hypersensitivity or allergies to one or more components. Do not use with other medical devices or medicines without first consulting a doctor or pharmacist.

Storage: keep in a cool and dry place, away from light and heat sources.

Packaging : 6 micro-enemas of 9 g each. The single-dose container allows the proper storage of the product.

Expiry date: See the expiry date reported on the box. The expiry date refers to product unopened and properly stored.

Date of last revision: January 2025

Pharmalife Research s.r.l.
Via Casa Paradiso 2/A
23846 Garbagnate Monastero (LC)
Italy

LEGEND

Manufacturer

Batch number

Keep away from light

Product code

Do not re-use

Expiry date

Compliance with European Union regulations

Attention: consult the instructions for use

Do not use in case of damaged pack