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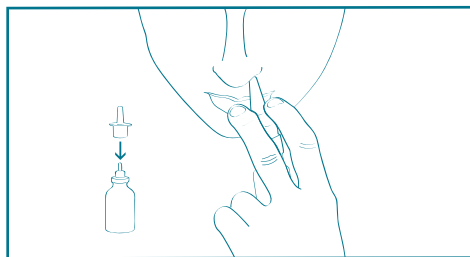
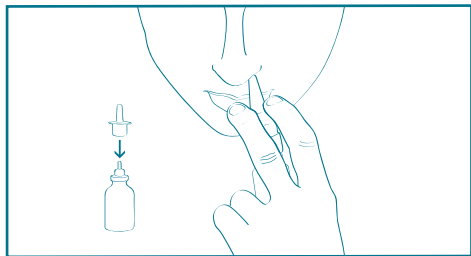
A 2.2% hypertonic solution containing sea water from Brittany, Aloe Vera and Blackcurrant Plant extracts, and Mountain Pine, Melaleuca (Tea Tree), and Lemon essential oils.

Use: the Benexè® Nasal Spray medical device is a nasal decongestant. It acts locally by osmosis, helping to reduce congestion, and indirectly, inflammation. Benexè® Nasal Spray is a hypertonic (2.2%) sea water solution enriched with plant extracts and essential oils from organically grown plants. Free from propellant gas.

Indications: Benexè® Nasal Spray is recommended as a first-line adjuvant therapy in the case of inflammatory processes of the upper airways, such as sinusitis, different types of colds, and allergic and seasonal rhinitis.

Dosage: 1-2 sprays per nostril up to 4 times per day.

Description: When brought into contact with blocked mucous membranes, the hypertonic solution causes the secretion of water which, drawn by osmosis, helps to dilute built-up thick mucus, facilitating its fluidification. The reduction of mucus viscosity improves mucociliary clearance and helps to remove viruses, bacteria, pollen, as well as macro and micro particles through the expulsion of mucus. By eliminating abnormal discharge and treating the mucous membrane with natural substances that create a hostile environment for bacterial (or microbial) growth, Benexè® Nasal Spray provides relief and prevents relapse.

Tolerability and side effects: the absence of vasoconstrictors and the natural action of organic plant extracts and essential oils, make Benexè® Nasal Spray a well-tolerated product that can be used often (not more than 15 days) without causing tolerance or "rhinitis medicamentosa". In rare cases, it may cause epistaxis. In rare cases, when there is a hypersensitivity to its components, it may cause a local allergic reaction. A slight tingling sensation may be experienced inside the nostril right after application. Should you experience side effects other than those described herewith, please contact the manufacturer immediately and consult your physician.

Instructions for use: blow your nose before using the spray. Spray the solution into each nostril quickly and energetically, breathing deeply and holding your head upright. After spraying, take a deep breath with your mouth closed, then slightly press the other nostril with your finger so that the solution is distributed onto the entire mucous membrane. After use, rinse the nozzle with warm water and dry it with a tissue.

Warnings and precautions.

- Use at room temperature (20°C - 25°C).
- Store the medical device in a dry place, away from direct sunlight.
- Do not expose the bottle to a heat source.
- Do not use on children under the age of 12.
- Keep out of children's reach.
- Avoid contact with the eyes.
- Do not use in case of known hypersensitivity to any of the components.
- Do not use in case of hypersensitivity to crustaceans and/or molluscs.
- Do not swallow.
- Do not use in ways and for purposes other than those specified by the manufacturer in these instructions for use. The bottle is strictly for personal use; it might carry infections.
- Do not use on damaged mucous membrane. Should you experience any side effects, please discontinue treatment.
- If used with other drugs/medical devices, consult your doctor in advance.
- Use within 1 month after opening the bottle.
- To date, the components of Benexè® Nasal Spray have no known contraindications when used during pregnancy or breastfeeding; in these cases, consult your doctor before using the product.

A multi-action product to deeply cleanse the nasal cavities with a decongestant effect.

Sea water (concentrate) = cleansing and softening action

The sea water contained in this formula comes from the sea near Brittany. Its composition is similar to our blood plasma and is particularly rich in mineral nutrients and trace elements. In addition to cleaning the nasal cavities and removing dust and pollen, hypertonic sea water favours mucociliary clearance and the fluidity of built-up mucus facilitating its expulsion.

Glyceric blackcurrant extract

Aloe vera dry extract

Mountain pine essential oil = fragrance and balsamic action

Organic melaleuca alternifolia (tea tree) essential oil

Organic lemon essential oil = fragrance

Disposal: the device must be disposed of in compliance with the waste management regulation of your local management authority. Do not dispose of the product or its packaging into the environment.